

HALOMIST™

Whole Room Disinfection System



All-Purpose Disinfectant

Healthcare-grade, Bactericide* | Virucide** | Fungicide
Contains Hydrogen Peroxide & Silver

When used in the HaloFogger®:

Sporicidal - Kills 99.9999% of Clostridium difficile spores

DryMist - No wipe, no rinse, non-corrosive

Easy to Apply - Reaches into every nook, crevice and corner that disinfecting sprays and wipes can't.

CAUTION

KEEP OUT OF REACH OF CHILDREN

See side panel for additional Precautionary Statements

Net Contents: 128 FL OZ (1 GAL)

Active Ingredients:

Hydrogen Peroxide	5.00%
Silver	0.01%
Other Ingredients	94.99%
TOTAL	100.00%

For use in:

All Medical, Healthcare, Semi-Critical Care, and Long Term Care environments.

Animal Industry, Bio-Safety labs, Biotech Industry, Commercial, Education, Entertainment, Government, Historic Buildings, Homeland Defense, Hospitality, Industrial, Institutional, Multifamily housing, Military, Pharmaceutical, Recreational, Entertainment and Residential Settings and Assets, Senior Living, Tissue banks, Transportation (public and private), Worship Facilities.

Food Handling Sites: Storage, Preparation, Processing, and Serving, Food Manufacturing Plant, Food Handling Establishments, Food Warehouses, Cafeteria, Farmer's Market, Food Service Establishment, Supermarket or Grocery Store. FOOD AREAS: In establishments where food and food products are held, prepared, processed and served, application may be made only when the facility is not in operation provided exposed food is covered or removed from the area being treated prior to application. Food contact surfaces must be rinsed with potable water after use of the product.

Questions? Comments? Call: 1-302-454-8102 or visit us at www.halosil.com. Made in USA Distributed by: Halosil International Inc, 91 Lukens Drive, New Castle, Delaware 19720 or an authorized Halosil International distributor. HaloMist and HaloFogger are registered trademarks of Halosil International

DIRECTIONS FOR USE It is a violation of Federal law to use this product in a manner inconsistent with the label.

HaloFogger: For use as a microbial disinfectant fogging solution in environments for disinfection of dry, pre-cleaned, hard, non-porous surfaces in sealed spaces and rooms located in healthcare facilities and settings. Do not deviate from standard cleaning regimes when using HaloMist. Use product only with HaloFogger dispensing device following detailed instructions provided in the HaloFogger User Manual. Read and follow directions in user manual on room preparation, room set up and treatment protocol, and equipment operating procedures for the HaloFogger device.

When used in HaloFogger, product effectively kills the following pathogens: **†Bacteria:** *Clostridium difficile* (spore form) (ATCC # 43598), *Pseudomonas aeruginosa* (*Pseudomonas*) (ATCC #15442), *Staphylococcus aureus* (*Staphylococcus*) (Staph) (ATCC 6538)

Special instructions for cleaning and decontamination against HIV-1 on precleaned environmental surfaces/ objects soiled with blood/body fluids:

In health care settings or other settings where there is an expected likelihood of soiling inanimate surfaces/ objects with blood or body fluids, and where the surfaces/ objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus Type 1 (HIV-1) (AIDS Virus) the following special procedures must be used. PERSONAL PROTECTION: When handling items soiled with blood or other body fluids wear protective equipment, including disposable latex or rubber gloves, gowns, masks, and eye coverings. CLEANING PROCEDURE: Blood or other body fluids must be thoroughly cleaned from the soiled surfaces and objects before product can be applied. CONTACT TIME: To kill HIV-1 (AIDS Virus), a contact time of 10 minutes at room temperature. DISPOSAL OF INFECTIOUS MATERIALS: Blood and other body fluids should be autoclaved and disposed of according to local regulations for infectious waste disposal.

Sprayer: Prior to disinfection or sanitization, pre-clean surface. For use on hard, non porous, nonfood contact surfaces only. Hold container upright 6" to 8" from surface. Spray 2 to 3 seconds until wet. **To Disinfect:** Let stand for ten minutes to air dry. Rinsing not required. Wipe with paper towel, clean dish towel or clean microfiber towel. **To Sanitize:** Let stand for 5 minutes to air dry. Rinsing not required. Wipe with paper towel, clean dish towel or clean microfiber towel. **To Deodorize:** Spray on surfaces as needed. Let air dry.

When used in Sprayer, product effectively kills the following pathogens: ***Bacteria:** *Pseudomonas aeruginosa* (*Pseudomonas*) (ATCC 15442), *Salmonella enterica* (*Salmonella*) (ATCC 10708), *Staphylococcus aureus* (*Staphylococcus*) (Staph) (ATCC 6538), *Methicillin Resistant Staphylococcus aureus* (*Staph*) (MRSA) (ATCC 33592), *Escherichia coli* (*E coli*) (ATCC 11229) ****Viruses:** Feline calicivirus (as surrogate for Norovirus) (ATCC VR-782), Human Immunodeficiency Virus type 1 (HIV) (Stain HTLV-IIIb), Influenza A strain Hong Kong (flu virus) (ATCC VR-544), Influenza A (H5N1) (influenza) (flu virus) (Strain VNHSN1-PR8/CDC-RG CDC #2006719965), Minute virus of Mice (MVM) (ATCC VR 1346), Norovirus [Norwalk virus] (ATCC VR-782), Rhinovirus type 37 (Rhinovirus) (ATCC BR-1147, Strain 151-1) **Fungi:** *Trichophyton mentagrophytes* (ATCC 9533), *Aspergillus niger* (ATCC 16404)

Effective sanitizer against the following (5 minute contact time): ***Bacteria:** *Proteus mirabilis* (ATCC 9240), *Staphylococcus aureus* (*Staphylococcus*) (Staph) (ATCC 6538), *Methicillin Resistant Staphylococcus aureus* (*Staph*) (MRSA) (ATCC 33592), *Enterobacter aerogenes* (ATCC 13048)

Special Instructions for Cleaning Prior to Disinfection against Clostridium difficile spores:

Personal Protection: Refer to label PPE requirements. Cleaning Procedure: Pre-cleaning is to include removal of fecal matter/waste from surfaces/objects. Infectious Materials Disposal: Materials used in the cleaning process that may contain feces/wastes are to be disposed of immediately in accordance with local regulations for infectious materials disposal.

PRECAUTIONARY STATEMENTS Hazards to Humans and Domestic Animals

CAUTION Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum or using tobacco.

FIRST AID If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a Poison Control Center or doctor for treatment advice. If on Skin: Rinse skin immediately with plenty of water for 15-20 minutes. The affected skin can turn white and itchy. These symptoms will disappear within a few minutes.

STORAGE AND DISPOSAL Do not contaminate water, food, or feed by storage and disposal. Keep out of direct sunlight and away from heat. Do not freeze. Store in original container in areas inaccessible to small children and pets. Non-refillable container. Do not reuse or refill this container. Recycle, if available, or discard in trash. If product leaking or spills should occur, please dilute with water and dry with absorbent material, or dilute with water as it is flushed into waste water drain.

Questions? Comments, or in case of an emergency, call toll free (1-302-454-8102). Have the product container or label with you when calling a Poison Control Center, or doctor, or going for treatment.